


### Control Locations

1. Snooze Button
2. Hour Button
3. Minute Button
4. Time Set Button
5. Alarm ON/OFF/SET Switch
6. Alarm SOUND/VIB/BOTH Switch

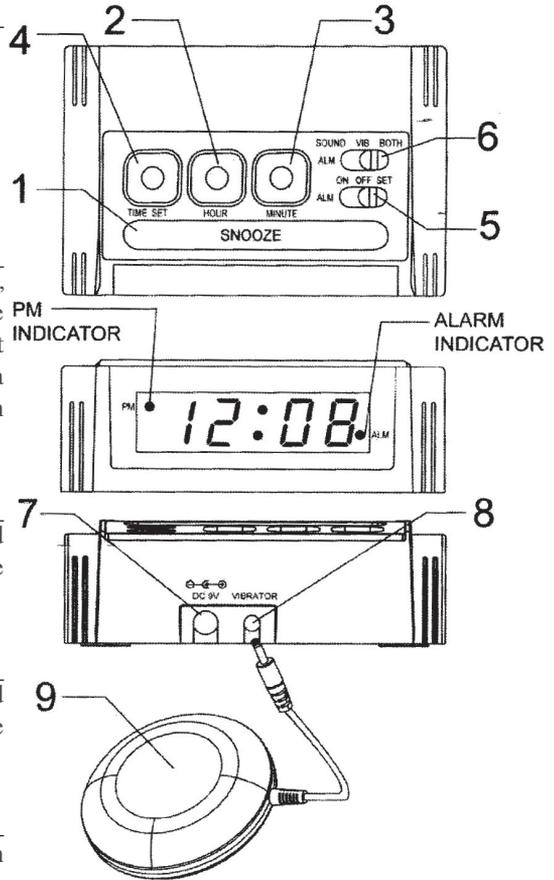

### Operation

Plug the adaptor into the power outlet, and connect to the clock (7). The entire display will flash meaning the incorrect time is being displayed. This is a normal condition and will stop when the correct time has been set.

### PM Indicator

A dot appears in the upper left hand corner of the display indicating that the time shown is PM.

### Alarm on Indicator

A dot appears in the lower right hand corner of the display indicating that the alarm has been set.

### Time Setting

Press and hold the TIME SET Button (4) and the HOUR Button (2) to advance the hour display. Then press and hold the TIME SET Button (4) and the MINUTE Button (3) to advance the minutes until the correct time is displayed. Be sure to set it correctly in the AM or PM mode. The PM Indicator will be "ON" when the clock is reading PM time and "OFF" when reading AM time.

### Alarm Time Setting

Move the ALARM ON/OFF/SET Switch (5) to SET position To set the alarm, press and hold the HOUR Button (2) and then press and hold the MINUTE Button (3) until the correct alarm time is displayed. Be sure to set it correctly in the AM or PM mode.

### To Operate Alarm (Sound and/or Vibration)

Move the ALARM SOUND/VIB/BOTH (6) to desired position - "SOUND" position for sound alarm only, "VIB" for vibration only, or "BOTH" position for sound and vibration alarm. The Alarm On Indicator will light up meaning the alarm is set at the preset time. To silence the alarm, move the ALARM ON/OFF/SET Switch (5) to the "OFF" position. Please connect the vibration pad (9) to the clock (8) for vibration alarm.

## **Snooze**

---

When the alarm sounds simply press the SNOOZE Button (1) to temporarily silence the alarm. The alarm will sound again 8-9 minutes later. This can be repeated again and again. To silence the alarm, move the ALARM ON/OFF/SET Switch (5) to the "OFF" position.

## **Battery Back-up (Battery not included)**

---

In the event of a power failure, this unit will automatically switch to a battery stand-by circuit when a battery is installed. Open the BATTERY COMPARTMENT on the bottom of the unit and install a 9-volt battery. Under battery operation, the clock will keep the time running and the alarm will sound at the preset time. When relying on back up battery power the time will not show on the display but the internal clock will continue to run.

## **Warning**

---

- This product is not designed to be exposed to any form of liquid
- The VibeAlert™ alarm clock remains readily operable when the plug is connected to a power outlet

## **Warranty**

---

Krown Manufacturing, Inc. warrants that the VibeAlert™ alarm clock and bed shaker sold by Krown Manufacturing, Inc. is free from defects in materials and workmanship under normal use and service for one year. This warranty is applicable only to the original purchaser of the VibeAlert™ alarm clock and bed shaker when accompanied by a sales receipt stating the date of purchase and the name of the company from which purchased. This warranty is in lieu of and excludes all other warranties expressed or implied including any implied warranty of merchantability or fitness any of any obligations on the part of Krown Manufacturing, Inc. shall either replace or repair the VibeAlert™ alarm clock and bed shaker at no cost to the original purchaser within one year of the date of purchase. During a one year period the purchaser may return the VibeAlert™ alarm clock to Krown Manufacturing, Inc. at the following address:

**Krown Manufacturing, Inc.**  
ATTN: Repair Department  
3408 Indale Road, Fort Worth, TX 76116

Shipping charges are at the customer's expense. Please include a check or money order made out to Krown Manufacturing, Inc. in the amount of \$7.50 to cover shipping and handling. This warranty does extend to any VibeAlert™ alarm clock and bed shaker made by Krown Manufacturing, Inc. which has been subjected to abuse, neglect, accident, improper applications, improper installation, or to use in violation of the instruction furnished by Krown Manufacturing, Inc. This warranty does not extend to or apply to any VibeAlert™ alarm clock and/or bed shaker which has been repaired or altered by any person who has not been expressly approved by Krown Manufacturing, Inc. Krown Manufacturing, Inc. shall not be liable for any special or consequential damages or for loss or expense directly or indirectly arising from use of the products or any inability to use them either separately or in combination with other equipment or product accessories or from any other cause. This warranty is only applicable to a product purchased through Krown Manufacturing, Inc. or Compu-TTY, Inc. dealer/distributor. Other than as stated herein, there are no other warranties whether by sample or model, by description or quantity, whether expressed or implied and there are no implied warranties of fitness for a particular purpose or of merchantability.

